SUPERMASK®

PROTECTIVE EYE MASK FOR HORSES

MASQUE OCULAIRE PROTECTEUR POUR CHEVAUX
MASCARA DE PROTECCION OCULAR PARA CABALLOS

NEW!

Money Saving Coupon
for **REPEL-X**
INSECTICIDE & REPELLENT Inside

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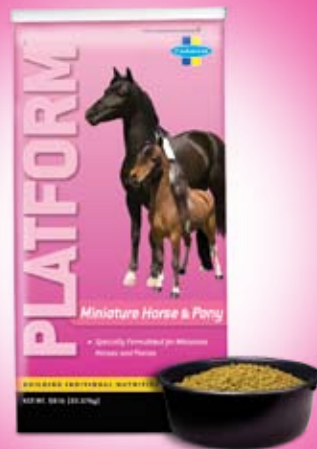
Other FARNAM products
available for miniature horses

PLATFORM®

Miniature Horse & Pony Feed

Specially formulated for smaller,
more efficient digestive systems of
miniature horses and ponies.

Visit platformfeed.com for more information!

- Keeps flies away from eyes without obstructing vision
- Protects eyes from flying dust, dirt and debris
- Aids healing after eye injuries
- Use as a sunscreen to protect eyes and face
- No rubbing or harsh abrasion
- Often lasts more than one season
- Fleece-lined ear opening and nose band conform snugly to keep flies out

LIMITED WARRANTY AND DISCLAIMER:
Buyer assumes all risks of use, storage and handling of this material not in strict accordance with label directions and usage. In no event shall Seller's liability exceed the purchase price of this product.

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It's Fly-Proof!

Protects eyes from:

- Fly irritation
- Flying dust, dirt, debris

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FARNAM HORSE PRODUCTS
A DIVISION OF FARNAM COMPANIES, INC.
P.O. BOX 34820, PHOENIX, AZ 85067-4820 USA

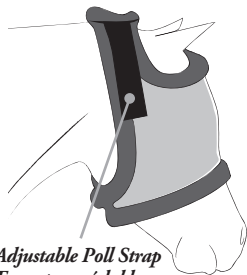
SUPERMASK[®]

MINIATURE HORSE FLY MASK

MASQUE ANTI-MOUCHS POUR CHEVAL MINIATURE MÁSCARA CONTRA LAS MOSCAS PARA CABALLO MINIATURA

Adjustable Poll Strap
Fermeture réglable
Correa de nuca ajustable

It really stays on the miniature horse! Plus, break-away throatlatch releases with pressure for the miniature horse's safety.

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Directions for Use: Start by opening hook and loop closure at throatlatch and at poll (behind ears). Gently place mask around miniature horse's face and attach hook and loop closure behind ears adjusting as required to fit face of miniature horse. Attach hook and loop strap under throatlatch and adjust as necessary to fit miniature horse. If there is excess loop (the upper strap) at throatlatch or poll area, please trim upper loop strip. Ensure fly mask is properly adjusted to fit miniature horse and is not too snug or too loose.

NOTE: When removing, undo the throatlatch gently. The ripping sound might startle your miniature horse and make him jump.

For Best Results:

- Remove mask nightly and check miniature horse's face and eye condition.
- Rinse mask weekly.
- Check fur trim periodically for trapped burrs or irritating foreign objects.

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©2007 Farnam Companies, Inc.